


TOPAZ[®] Control Panel Reference Card

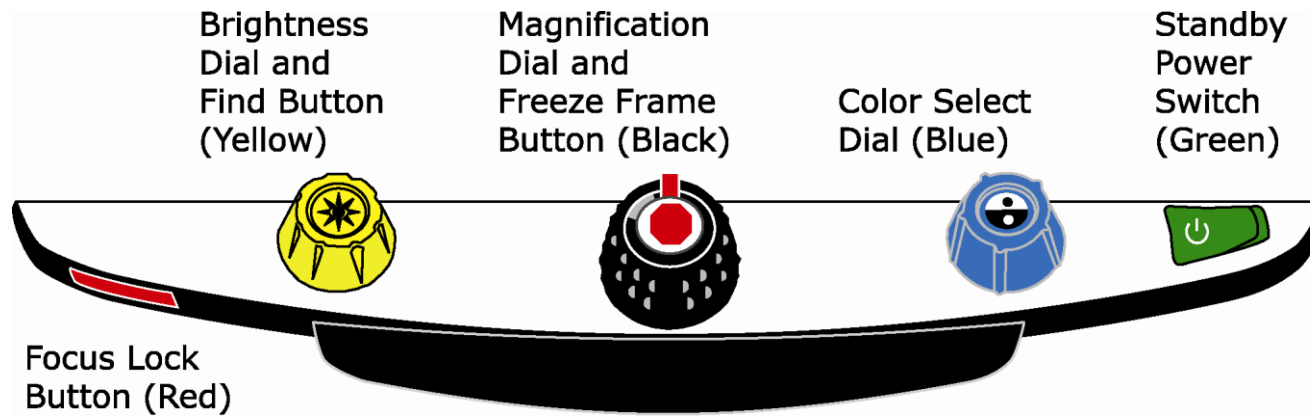

| Dial / Button / Switch | Functionality |
|--|---|
| Brightness Dial / Find Button (Yellow) | <ul style="list-style-type: none"> • Turn to increase or decrease screen brightness. • Press and hold center to zoom out using the Find feature. Release to zoom back in. |
| Magnification Dial / Freeze Frame Button (Black) | <ul style="list-style-type: none"> • Turn to increase or decrease magnification. • Press center to switch Freeze Frame on or off. |
| Color Select Dial (Blue) | <ul style="list-style-type: none"> • Turn to choose a color mode that best suits your vision needs. • Press center to switch the horizontal reading line on or off. |
| Standby Power Switch (Green) | <ul style="list-style-type: none"> • Press to the right to place TOPAZ in standby mode. Press to the left to turn on TOPAZ from standby mode. |
| Focus Lock Button (Red) | <ul style="list-style-type: none"> • Press to switch Focus Lock on or off. • Press and hold simultaneously with the center of the yellow Brightness dial to switch LED reading lights on and off. |